VET EXPERT
BASED ON EVIDENCE

Vet Expert Rapid Test CDV Ag

EN PRINCIPLES

The Vet Expert CDV Ag is a chromatographic immunoassay for the qualitative detection of Canine Distemper Virus antigen in conjunctival swab, urine, serum or plasma. Letters T and C on the surface of the plate indicate test line (T) and control line (C). The control line (C) is used for procedural control and should always appear if the test procedure is being performed properly and the test reagents of the control line are working correctly. A colored test line (T) will be visible in the result window if the target antigens are present in the sample.

Sensitivity: 98.8%
Specificity: 97.7%

MATERIALS PROVIDED (5/10 TESTS/KIT)

- 5/10 Vet Expert CDV Ag test plate(s)
- 5/10 tubes containing assay diluent
- 5/10 collection swabs
- 5/10 disposable pipettes
- Instruction for use

PRECAUTIONS:

- For veterinary diagnostics only.
- For best results, strict adherence to the instructions is required.
- The test plate is sensitive to humidity and heat. Perform the test immediately after opening the pouch.
- The test components are single-use only.
- Do not touch the membrane in the result window.
- Do not use the test kit beyond the stated expiration date marked on the package label.
- Do not use the test kit if the pouch is damaged.
- Do not mix components from different lot numbers.
- Handle with the sample as potentially infectious.
- Decontaminate and dispose of all samples and materials in accordance with national and local regulations.
- Consider the result results as invalid after the specified read out time.

STORAGE OF THE TEST KIT

The test kit must be stored between 2-30°C. The test is stable within the expiration date. DO NOT FREEZE. Do not expose the test kit to sunlight.

SPECIMEN COLLECTION AND PREPARATION

Conjunctival swab, urine, serum or plasma are the appropriate specimens for this test. After collection, the specimen should be immediately extracted and tested. If specimens are not immediately tested, they should be refrigerated at 2-8°C up to 48 hours. For longer storage freeze the specimen at -20°C or below.

PROCEDURE OF THE TEST (LOOK AT THE TOP)

- Collect the sample from conjunctival sac or urine using the swab. In case of serum or plasma samples use the pipette.
- Insert the swab into the tube containing assay diluent and stir with the swab for at least 10 seconds. In case of serum or plasma samples, add 2-3 drops of the sample to the specimen tube and mix well.
- Remove the test plate from the foil pouch and place it on a flat and dry surface.
- Add four (4) drops of the mixed sample into the sample well using the pipette.
- The sample will flow across the result window. If the migration has not appeared after 1 minute, add one more drop of the mixed sample to the sample well. Read the test results after 5-10 minutes.

INTERPRETATION OF THE TEST (LOOK AT THE BOTTOM)

- Negative result – only control line (C) appears in the result window
- Positive result – both test line (T) and control line (C) appear in the result window – indicates the presence of target antigen in the sample
- Invalid result – if the control line (C) does not appear, the result might be considered invalid. The sample should be retested.

LIMITATION OF THE TEST

Although the Vet Expert CDV Ag is very accurate, a low incidence of false results can occur. In case of questionable results, repeat tests should be performed. Definitive clinical diagnosis should not be based on the results of a single test, but must be an effect of extended clinical and laboratory analysis.

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