## Vet Expert Rapid Test Total IgE Ab

### EN PRINCIPLES

The Vet Expert Total IgE is a chromatographic immunoassay for the qualitative detection of total IgE antibody in canine serum. Letters T and C on the surface of the test plate indicate test line (T) and control line (C). The control line (C) is used for procedural control and should always appear if the test procedure is being performed properly and the test reagents are working correctly. A colored test line (T) will be visible in the result window if there are enough IgE in the specimen.

Sensitivity: 97.1%  
Specificity: 96.4%

### MATERIALS PROVIDED (5 TESTS/KIT)

- 1) Vet Expert Total IgE test kit
- 2) 5 tubes containing assay diluent
- 3) 5 disposable pipettes (10 µl, marked on the pipette)
- 4) 5 disposable pipettes (regular)
- 5) Instruction for use

### PRECAUTIONS

- 1) For veterinary diagnostics use only.
- 2) For best results, strict adherence to the instructions is required.
- 3) The test plate is sensitive to humidity and heat. Perform the test immediately after opening the pouch.
- 4) The test components are single-use only.
- 5) Do not touch the membrane in the result window.
- 6) Do not use the test kit beyond the stated expiration date marked on the package label.
- 7) Do not use the test kit if the pouch is damaged.
- 8) Do not mix components from different lot numbers.
- 9) Handle with the sample as potentially infectious.
- 10) Decontaminate and dispose of all samples and materials in accordance with national and local regulations.
- 11) The result of the test is invalid after the specified read out time.

### STORAGE OF THE TEST KIT

The test kit must be stored between 2-30°C. The test is stable within the expiration date. DO NOT FREEZE. Do not expose the test to direct sunlight.

### SPECIMEN COLLECTION AND PREPARATION

Canine serum is appropriate for this test. Collect the whole blood into the collection tube (NOT containing anticoagulants), leave to settle for 30 minutes and then centrifuge blood to obtain serum. Samples should be stored at 2-8°C. For longer storage, freeze the samples at -20°C or below. Avoid repeated freezing and thawing.

### PROCEDURE OF THE TEST (LOOK AT THE TOP)

- 1) Remove the test plate from the foil pouch and place it on a flat and dry surface.
- 2) Using a provided pipette, add 10 µl (marked on the pipette) of canine serum to the tube containing assay diluent.
- 3) Seal the tube and mix the content thoroughly by inverting for a few seconds.
- 4) Using the pipette, add 4 drops of mixed sample into the sample well.
- 5) The sample will flow across the result window. If it does not appear after 1 min, add one more drop of the sample to the sample well. You can read the test results after 10 min.

### INTERPRETATION OF THE TEST (LOOK AT THE TOP)

- 1) Negative result – only control (C) line appears in the result window. Canine IgE concentration is below 10 µg/ml.
- 2) Positive result – both test (T) and control (C) lines appear in the result window. Canine IgE concentration is above 10 µg/ml, which is the minimum indicative and warning level of total IgE for allergic disease.
- 3) Invalid result – if the control (C) line does not appear, the result might be considered invalid. The sample should be retested.

### LIMITATIONS OF THE TEST

Negative result means that total IgE concentration is below 10 µg/ml in canine serum. It does not necessarily mean that the dog is free of an allergy. Steroids and antihistamine medications may affect test result, therefore 2 weeks withdrawal period is required prior the test. Parasite infection should be cleared before the test. In case of questionable results, other tests should be performed. Definitive clinical diagnosis should not be based on the results of a single test, but must be an effect of extended clinical and laboratory analysis.

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